

“This is a controlled document in accordance with HPA Document Control System. If printed off the website it is no longer guaranteed to be the latest version”.

Recommended Protocol for HBV genotyping and for antiviral resistance analysis.

This amplification protocol is based on a nested PCR. The final amplicon is approximately 1kb and will cover the entire HBsAg region as well as domains A to E of the overlapping polymerase region.

All primer sequences shown are 5'-3'.

Outer sense:	HBV Z - AGC CCT CAG GCT CAG GGC ATA
Outer antisense:	HBV 3 – CGT TGC CKD GCA ACS GGG TAA AGG
Inner sense:	HBV P - TCA TCC TCA GGC CAT GCA GT
Inner antisense:	HBV M- GAC ACA CTT TCC AAT CAA TNG G

First Round PCR

PCR Master mix per sample: (All reagents are from Invitrogen)

2.5µl	10XPCR Buffer
0.75µl	50mM Mg Cl ₂
0.5µl	10mM dNTPS
0.1µl	Taq Polymerase
0.5µl	HBV Z (20pmol/µl)
0.5 µl	HBV 3 (20pmol/µl)
15.15µl	dH ₂ O

Add 5µl viral DNA to 20µl PCR Master Mix.

Cycling conditions

95 ⁰ C	5 Minutes	}	34 Cycles
94 ⁰ C	30 Seconds		
55 ⁰ C	30 Seconds		
72 ⁰ C	1 Minute		
72 ⁰ C	2 Minutes		

“This is a controlled document in accordance with HPA Document Control System. If printed off the website it is no longer guaranteed to be the latest version”.

Second Round PCR

PCR Master mix per sample:

5µl	10XPCR Buffer
1.5µl	50mM Mg Cl ₂
1µl	10mM dNTPS
0.2µl	Taq Polymerase
1µl	HBV P (20pmol/µl)
1µl	HBV M (20pmol/µl)
39.3µl	dH ₂ O

8.3 Add 1µl first round product to 49µl PCR Master Mix.

Cycling conditions

95 ⁰ C	5 Minutes	}	34 Cycles
94 ⁰ C	30 Seconds		
50 ⁰ C	30 Seconds		
72 ⁰ C	1 Minute		
72 ⁰ C	7 Minutes		

Sequencing PCR

Sequencing reactions should be set up in accordance with the instructions provided by the manufacturer.

NB – Each sample second round amplicon is analysed using each of the following primers in individual reactions:

HBV P - TCA TCC TCA GGC CAT GCA GT

HBV M- GAC ACA CTT TCC AAT CAA TNG G


HBV N- ACTGAGCCAGGAGAAACGGACTGAGGC

HBV H- TATCAAGGAATTCTGCCCGTTTGTCTT

Comment

The use of a nested approach facilitates sequencing of low level HBV DNA such as is found in the anti-HBe seropositive individual. The four sequencing reactions allow secure sequence data for assembly of the small HBs contig.

“This is a controlled document in accordance with HPA Document Control System. If printed off the website it is no longer guaranteed to be the latest version”.

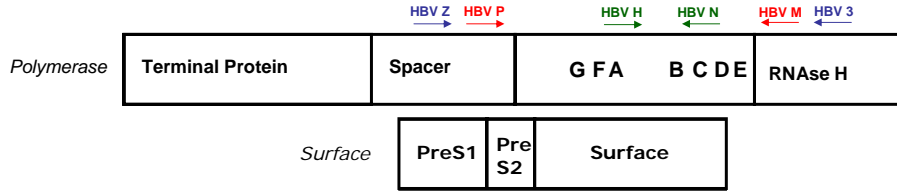

—→ Outer primers

—→ Inner primers

—→ Additional sequencing primers

Primer Name	Position (numbered in accordance with Pugh et al., 1986)
HBV Z	1179-1199
HBV 3	2478-2455
HBV P	1292-1311
HBV M	2306-2287
HBV N	1991-1965
HBV H	1767-1793